**all on**<sup>®</sup>

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DK

# Product description

The All On® mattresses are designed to give complete relief in prevention and treatment of pressure ulcers.

The mattresses ensures stability without hindering the patient's movements. Even with the slightest movement, the cushion will stimulate the microcirculation of the skin, which optimises circulation.

All On® is made of a non-woven material, which makes the device soft, skin-friendly, heat and moisture absorbent, providing comfortable, optimal pressure relief.

## Indications / Purpose

Decubitus prophylaxis and therapy.  
Existing pressure ulcer.  
Decubitus risk.

## Application Risks /

## Contraindications

All On® can be used in beds, trolleys or bed bath trolleys with or without adjustable reclining surfaces. When using the All On® in beds with adjustable surfaces.  
Be sure to follow guidelines to avoid shear.

## Permissible operating conditions / locations

All On® can be used everywhere on the body.  
All On® is designed as a 'single-patient' only product and is therefore not to be reused on another patient.  
Single-patient use eliminates the risk of a possible bacterial spread between patients.

# Cleaning

## Note!

Do not use caustic or abrasive cleaners or sharp objects to clean All On®.

Use and washing results in wear on the outer fiber layer.

If the outer fiber layer is no longer intact or if Seat All® loses air, All On® can not be used and disposed of with ordinary waste \*.

## Cleaning of All On®:

Let out the air before washing.

All On® can be cleaned with water and soap. Remove stubborn stains with a damp cloth, use a mild anti-allergic detergent.

Before using All On®, it must be completely dry. Do not dry it in a tumble dryer.

## All On® disinfection:

If All On® has been contaminated by body fluids, it can be disinfected to prevent infection.

All On® can be sprayed with a suitable disinfectant or with alcohol / propyl alcohol on a damp cloth for wiping.

Before using All On®, it must be completely dry. Do not dry it in a tumble dryer.

## Guarantee

\* The warranty is limited and covers only manufacturing defects. A non-intended use of the product will invalidate the warranty.  
Normal wear is excluded from the warranty.

## Safety regarding All On Carry Mattress

### Attention!

Inspect all seams and handles and make sure that everything is working properly BEFORE the mattress is lifted.

# Inflation and adjustment

See pictograms at the back

The All On® mattresses is designed to prevent and treat pressure ulcers.  
The mattresses are for single patient use only.  
Please also ensure that the treated area is examined regularly and in accordance with the applicable clinical guidelines.  
When using Seat all with other products, please read the instructions for the other products.

Carefully remove the mattress from its outer packaging.  
Carefully unfold it, so the blue valves are facing upwards on every section of the mattress. (FIG. 1)

## To inflate:

Start the inflation in one of the end sections of the mattress.

Hold the device valve between the finger and thumb (D).  
Insert the pump that comes with the product (D) in the blue valve (FIG. 1).

When pumping, You must count the strokes when pressing the piston in (F).

**70052 All On® Overlay Mattress:**  
**Pump 45 strokes into each section. (FIG. 2-3-4)**

**70051 All On® Trolley Mattress:**  
**Pump 33 strokes into each section. (FIG. 2-3-4)**

**70064 All On® Carry Mattress:**  
**Pump 33 strokes into each section. (FIG. 2-3-4)**

This can be done with or without a person lying on the mattress.

**IMPORTANT** The All On® mattress must not be over inflated.

For optimal and comfortable positioning it should be soft and pliable, but not hard.

The section is now inflated to a recommended starting level.

Check the lifting height, The body should be approximately 25 mm (two fingers) above the supporting surface (Figure 5) Adjust if necessary by releasing or adding air..

The section should be inflated to a level to suit the needs of your patient.

The independent sections make it possible to optimize the air level to perfectly suit the patient (FIG. 7), see "important note" below.

Once you have inflated and adjusted the mattress to the required level, close the valve (FIG. 4) and press the blue circular adhesive patch (C), over the valve cap (FIG. 6).

## To deflate:

Hold the device valve between finger and thumb (E), insert the small blue pump nozzle (A) to release the air.

When using a Levabo manual pump the pump nozzle (A) is not needed because a vent button is built into the pump (E).

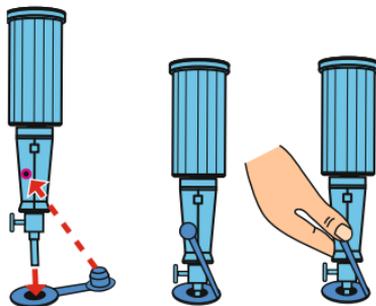
### IMPORTANT NOTE:

The All On® mattress must not be over inflated. For optimal and comfortable positioning it should be soft and pliable, but not hard.  
The body should be approximately 25 mm (two fingers) above the supporting surface (Figure 5!).

Check whether the lifting height is correct on a daily basis.

## Tip before inflation

Insert the pump into the valve.  
Insert the closure plug into the hole on the pump handle.  
Pump up.

If there is no hole, you can hold the closure plug with a finger.

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FR

# All On® -Range of mattresses

Product name	Size in cm.	Pump strokes for inflation	Packaging	Item no.	Weight	Air cells	Operating and storage	Material
All On® Overlay Mattress Single Patient - No cover	80 x 200	45	The All On® comes in an air-tight sealed package which includes a valve set, and Pump.	70052	1016 g	3	0-40° C	100% PE
All On® Trolley Mattress Single Patient - No cover	65 x 190	33		70051	850 g	3		
All On® Carry Mattress Single Patient - No cover	65 x 190	33		70064	1095 g	3		
All On® Overlay Mattress Reuseable with cover	80 x 200	45		70108	2500 g	3		TPU inner mattress
All On® Overlay Mattress Single Patient - with cover	80 x 200	45		70106	2500 g	3		100% PE

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Explanation of the symbols on the packaging				
	Read the instructions		Expiry date	CE Conformité Européenne
	Batch number		Single patient product	Material in the packaging
	Manufacturer		Manufacturing date	REF Order number
	Authorized representative in the European Community			

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## Declaration of Conformity (CE)

The company Levabo ApS, Sverigesvej 20A, 8660 Skanderborg, declares that the products are designed and manufactured in accordance with the technical documentation and legal requirements.

# Cover for All On Mattress: Approvals and regulations

Material Upper part: Medical BI ELASTIC PES PU

Material lower part: Tarpaulin PVC

Antibacterial and Antifungal:

ISO 22196:2011

PASS

Flammability BS 7175 ignition source 5

Pre-treatment: None

Method of Test: BS 7175: 1989 (2013) Section 3

PASS

Flammability BS 7177: 2008 +

A1: 2011 Low Hazard

Method of Test: BS EN 597-1:

1995 & BS EN 597-2: 1995

PASS

EN 1021-1-2:

PASS